
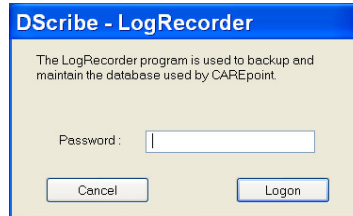
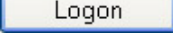



B.2 Quick Guide #2: Changing the CD

(Refer to CAREpoint Users Manual Section 7 for more information)


1. Obtain a blank CD-R disk to place in the system. **DO NOT** continue until you have the blank CD-R in hand.
2. Press  in the task bar at the bottom of the screen.



3. Enter the Password (factory default is **logrecorder**), & press .

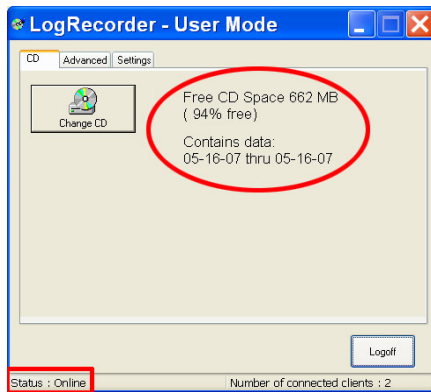


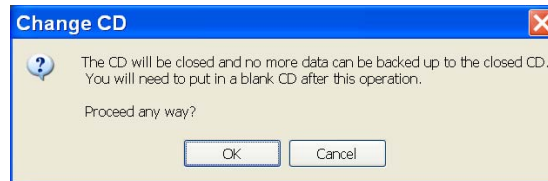
Wait until Status is "Online"...

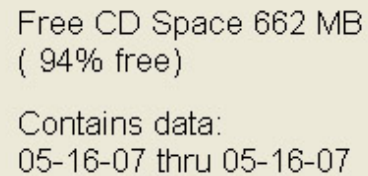


Free CD Space 662 MB
(94% free)
Contains data:
05-16-07 thru 05-16-07



And then the current CD Free percentage and the date range of the call audio and data from the CD will display.

4. Press . A dialog will display asking you to confirm that the CD will be closed and telling you that you will need to put in a blank CD.

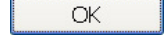

5. Press .

6. Wait for process to finish (Note: This may take several minutes).

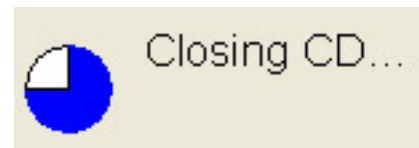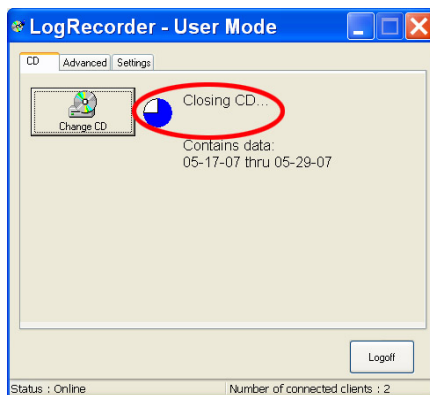

This will continue to display until the next dialog appears. See step 6.

7. A dialog instructing you to remove the CD and label it with the specified date range will appear. **DO NOT press OK yet!**




DO NOT press OK yet!

8. Remove the old CD from the drive tray and label it with the displayed date range.
9. Insert a new blank CD-R in the CD tray.


10. Press .

11. LogRecorder will scan the blank CD. Verify that the Free CD space is 100%.

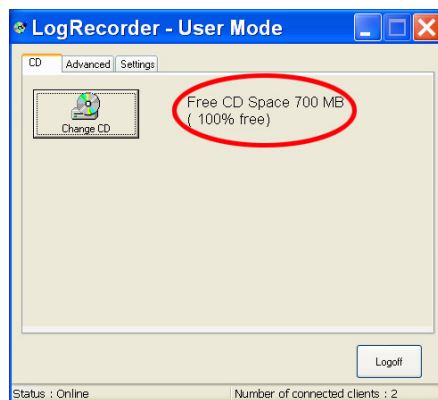

Free CD Space 700 MB
(100% free)

The size (in MB) of the CD may vary.
However, it should say 100% free.

12. If the Free CD space is 100%, press .

End.